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**KaWe**

<b>de</b>	Gebrauchsanweisung .....	2
<b>en</b>	User's Manual .....	8
<b>fr</b>	Mode d'emploi .....	14
<b>it</b>	Istruzioni per l'uso .....	20
<b>es</b>	Instrucciones de empleo .....	26
<b>pt</b>	Manual de operação .....	32
<b>ru</b>	Руководство по применению .....	38

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
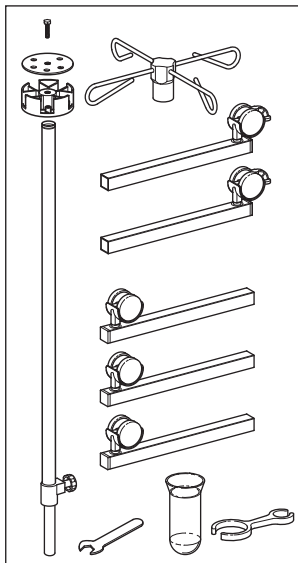
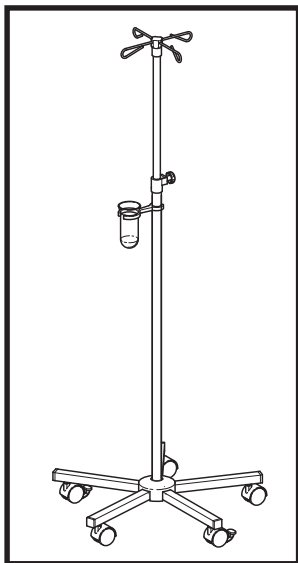
# User's Manual


## Stainless steel/chrome-plated infusion stand


Mounting parts

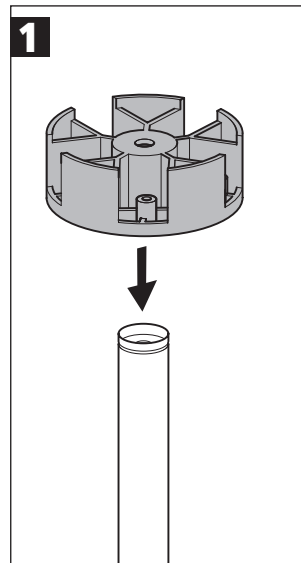1. Attach the foot juncture piece.

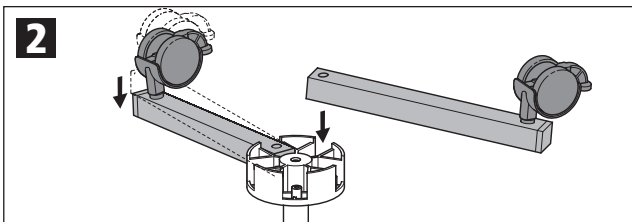
2. Attach two foot pieces with brake rollers.

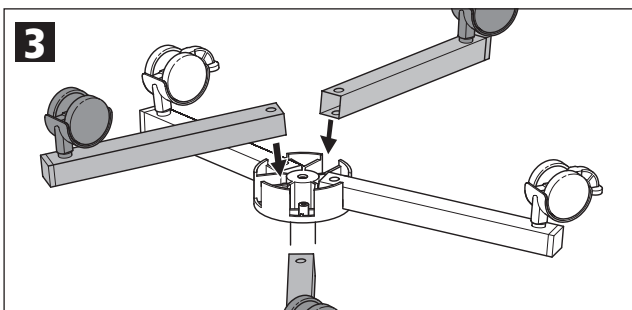
3. Attach three foot pieces without brake rollers.

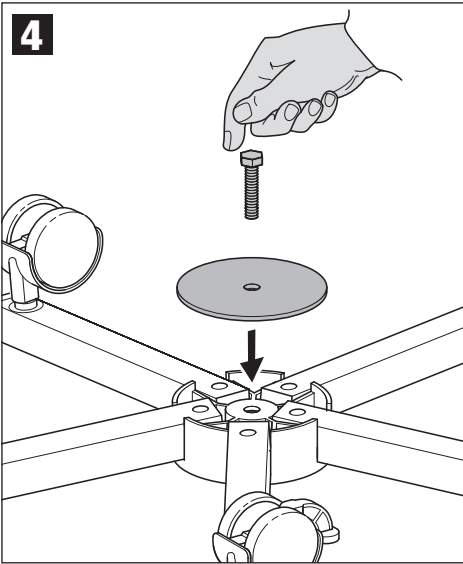
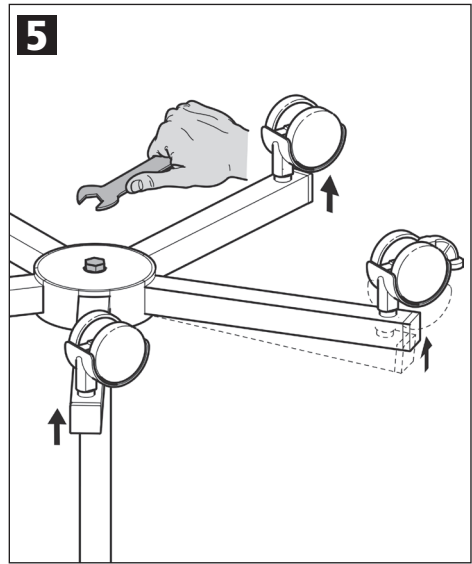
4. Screw in hexagon screws.

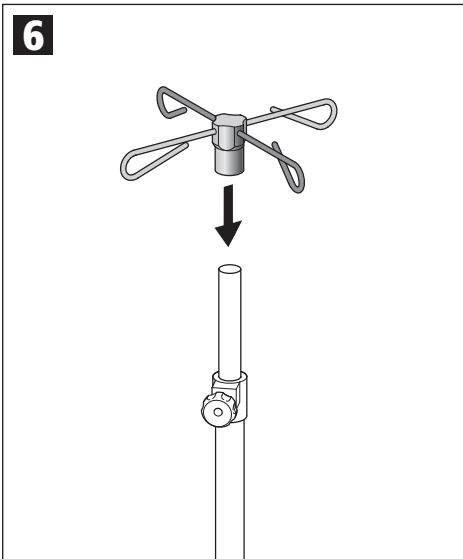
5. Tighten hexagon screws.

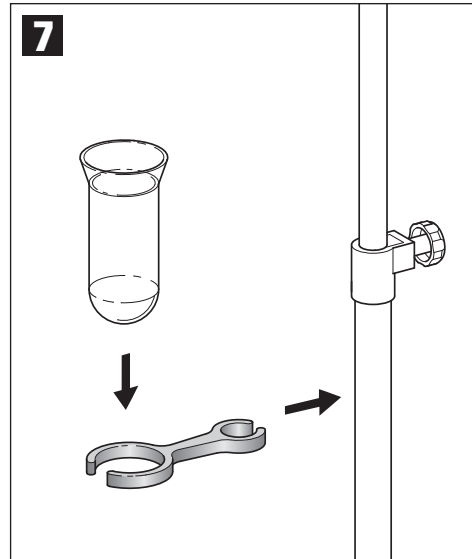
6. Mount the stainless steel safety hooks.

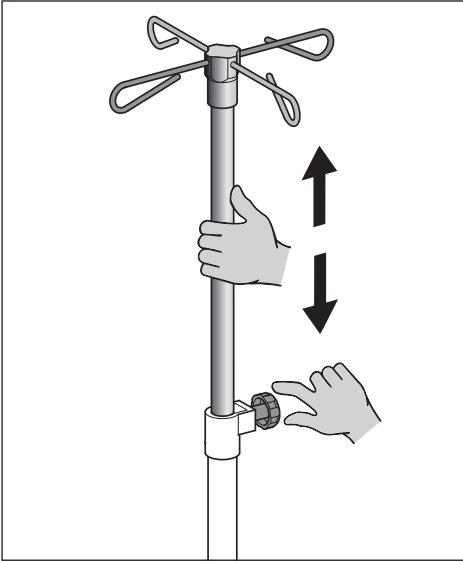
7. Push in glass drip dish attachment and attach glass drip dish.

Set the required height.


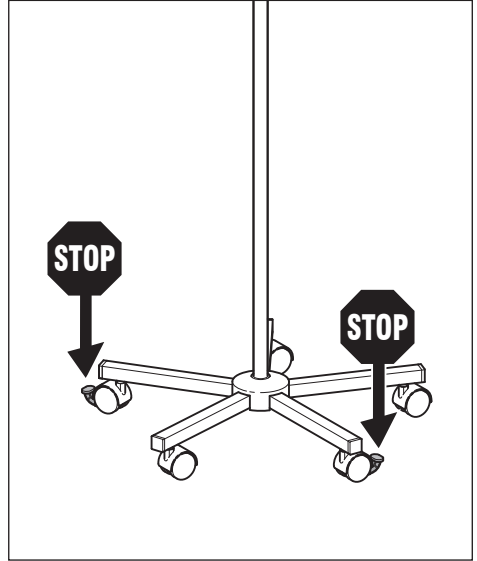
Adjust the brake rollers.

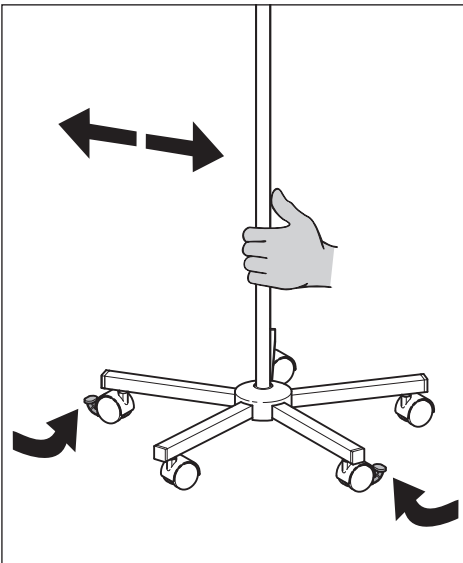
↔ Prior to moving the infusion stand, release the brake rollers.

Symbol key:


	Manufacturer
	Date of manufacture
	Lot code
	Heed the User's Manual
	Caution!
	Product reference number
	GOST-R certification for exports to Russia

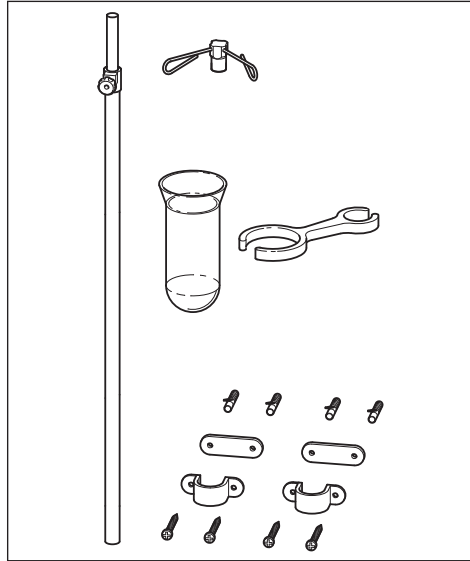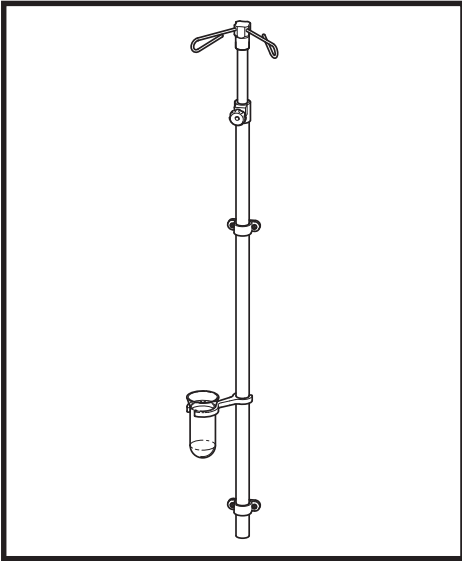Mounting parts

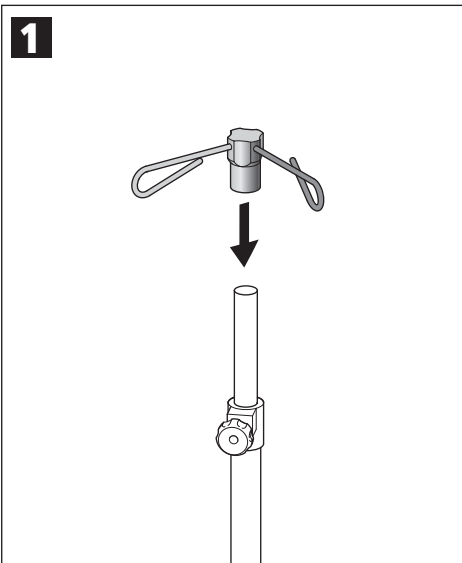
1. Attach safety hooks

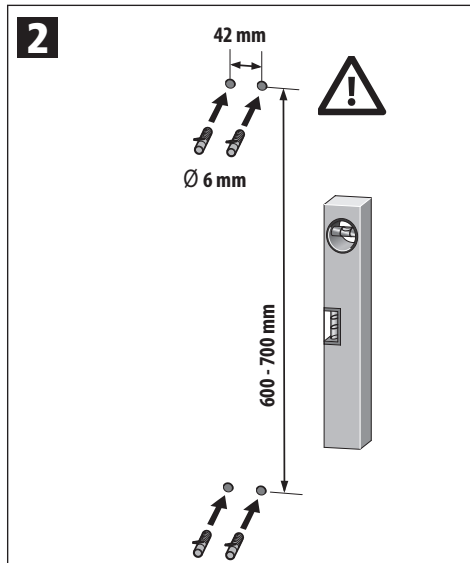
2. Drill four holes and insert anchors.

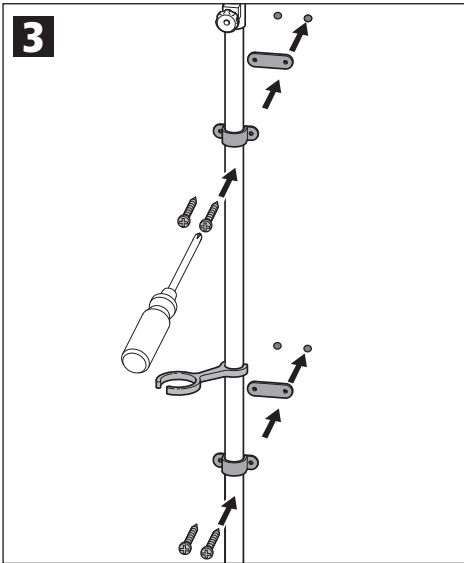
*Caution!* Check wall for power supply line, gas and water pipes.

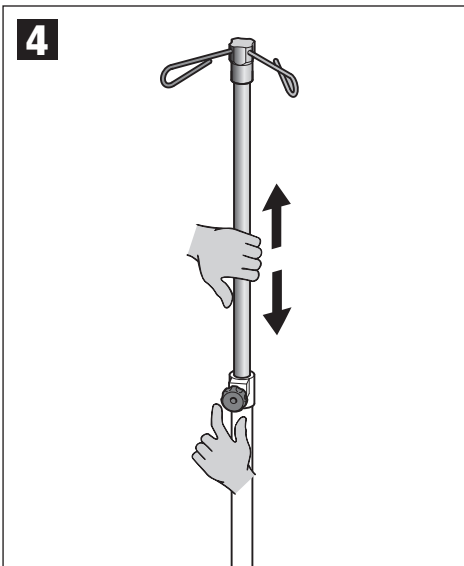3. Push in glass drip dish attachment and mount infusion stand.

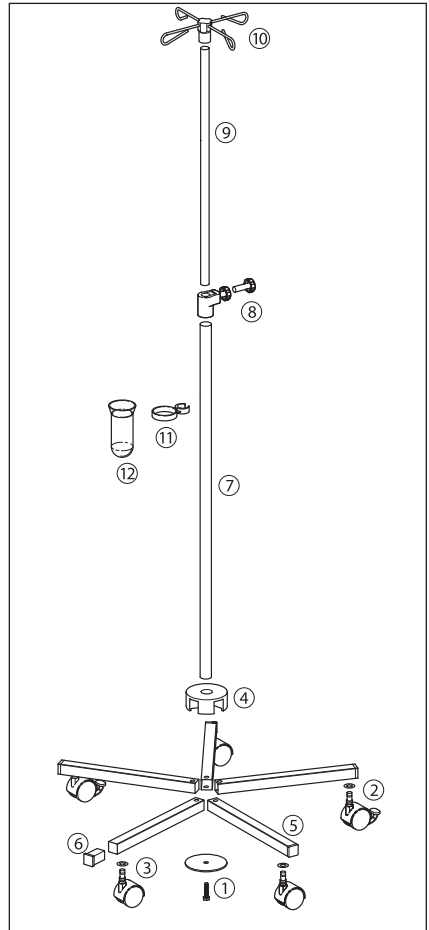
4. Set the required height.

#### Included:

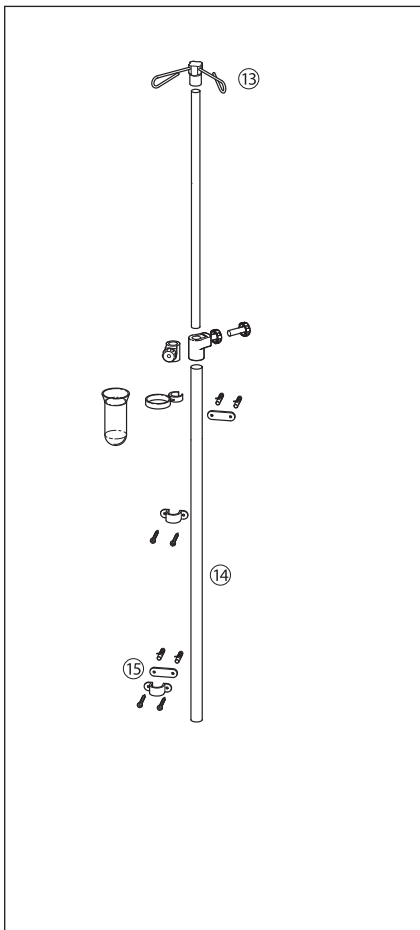- |   |   |
|---|---|
| 1 | fastening disc including screw                        |
| 2 | wheel - breaking wheel                                |
| 3 | wheel - safety wheel<br>(one is an anti-static wheel) |
| 4 | foot juncture, black                                  |
| 5 | rectangle tube  |
| 6 | end cap, black  |
| 7 | tube for stand  |
| 8 | clamping unit, black                                  |

- 9 telescoping stand post
- 10 standard safety hooks, black
- 11 holder for drip glass
- 12 drip glass
- 13 safety hook
- 14 telescoping stand post
- 15 plastic mounting bracket including screws and wall anchors

**Purpose:** Holding stand for infusion bottles – infusion bottles are allowed to have a maximum weight of 2 kg per suspension!

**Guarantee:** When used properly and with attention to this User's Manual we guarantee this product for two years after the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer!

**! Safety instructions!**

- The infusion stand is only to be used in good order and condition.
- The infusion stand is only to be used on flat surfaces. It must be ensured that the brake rollers are locked. Stability is ensured with a load of 2 kg and a sloped position of up to 10%.
- Prior to moving the infusion stand it must be ensured that the brake rollers are released.
- The 4-hook suspension piece is only to be used for infusion bottles.
- The user is liable for damages resulting from any use that is not in accordance with the intended use of the unit.

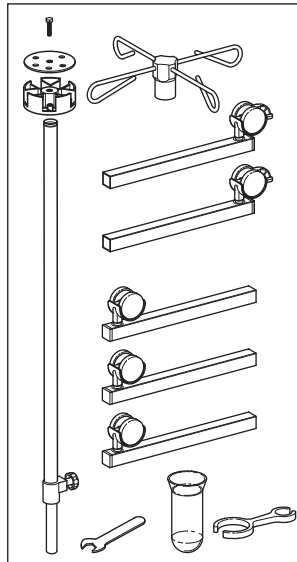
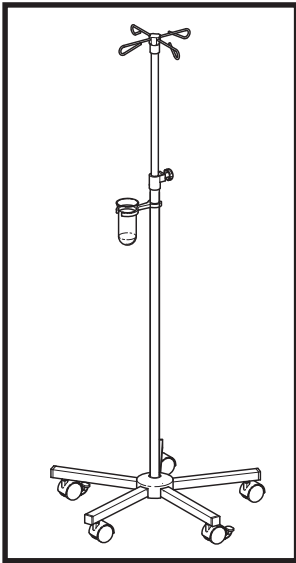
**Technical data:**

Maximum height	approx. 210 cm
Stand foot diameter	approx. 60 cm
Weight infusion stand stainless steel	approx. 2.9 kg
Weight infusion stand stand chrome-plated	approx. 2.6 kg


**Cleaning:** 1. Use alcohol-based cleaning agents. 2. Do not use any acids. 3. Manual cleaning with e.g. Helipur, Baktolan or Sagrotan in accordance with the instructions of the manufacturers. 4. Dirt that has accumulated on surfaces must be removed immediately. 5. Dirt that has accumulated on the rollers must be removed in order to avoid static charge. 6. Automatic cleaning is not possible. 7. No sterilization required.

**Maintenance:** 1. Regular visual and performance check of all component parts – defective parts must be replaced. 2. If necessary, re-tighten the screws.

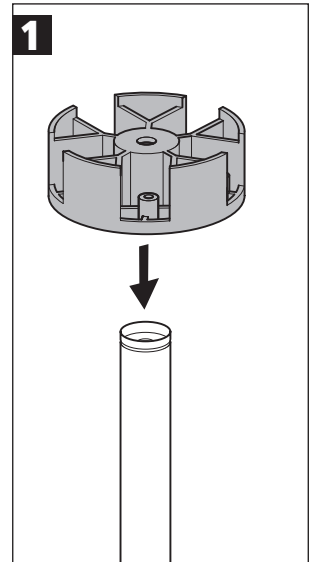Subject to technical modifications without prior notice.

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**KIRCHNER & WILHELM** GmbH + Co. KG